


Aug 22, 2023

CAMbank: SST Field Processing v1

DOI

<https://dx.doi.org/10.17504/protocols.io.81wgbxxo1lpk/v1>

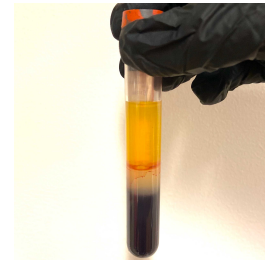

Elijah Overbey^{1,2}, Krista Ryon¹, JangKeun Kim¹, Christopher Mason^{1,2}

¹Weill Cornell Medicine; ²BioAstra

Cornell Aerospace Medi...



Elijah G Overbey

University of Austin (UATX), Weill Cornell Medicine, BioAstr...

Create & collaborate more with a free account

Edit and publish protocols, collaborate in communities, share insights through comments, and track progress with run records.

Create free account

OPEN  ACCESS



DOI: <https://dx.doi.org/10.17504/protocols.io.81wgbxxo1lpk/v1>

Protocol Citation: Elijah Overbey, Krista Ryon, JangKeun Kim, Christopher Mason 2023. CAMbank: SST Field Processing v1. protocols.io <https://dx.doi.org/10.17504/protocols.io.81wgbxxo1lpk/v1>

License: This is an open access protocol distributed under the terms of the **Creative Commons Attribution License**, which permits unrestricted use, distribution, and reproduction in any medium, provided the original author and source are credited

Protocol status: Working

We use this protocol and it's working

Created: August 22, 2023

Last Modified: June 27, 2024

Protocol Integer ID: 86809

Keywords: CAMbank, SST, serum, astronaut, SOMA, sst field processing v1 field processing, cornell aerospace medicine biobank, sst vacutainer, cambank, rbc pellet, serum, processing

Funders Acknowledgements:

WorldQuant Foundation

NASA

Grant ID: NNX14AH50G

NASA

Grant ID: NNX17AB26G

NASA

Grant ID: 80NSSC22K0254

NASA

Grant ID: NNH18ZTT001N-FG2

NASA

Grant ID: 80NSSC22K0254

NASA

Grant ID: NNX16AO69A

NASA

Grant ID: 80NSSC21K0316

Abstract

Field processing of SST vacutainers for the Cornell Aerospace Medicine Biobank (CAMbank).

Instructions for preserving: Serum and RBC Pellets.

Materials

Tube Type: BD Vacutainer® Serum Separator Tubes (SST) (BD Biosciences: #367987)



Perform Venipuncture

5m

- 1 After venipuncture, invert the tubes gently 8 to 10 times to fully mix tube anticoagulant with blood sample.

5m

Store the tube upright at room temperature until centrifugation.

Allow the blood to clot in an upright position for at least 30 minutes.

For optimal biochemical results, centrifuge and aliquot SSTs within 3 hours of venipuncture.

Centrifuge Settings

15m

- 2 Note: A **swing bucket** centrifuge is required.

3m

Set centrifuge:

- acceleration: 9
- deceleration: 9
- temperature: RT
- duration: 10 minutes
- speed: 1300xg

- 3 Place the SSTs in the centrifuge.

12m

Place a protective cover over the swing buckets in case of tube leakage.

Start the centrifuge.

Stand by the centrifuge until the centrifuge reaches max speed. Listen for signs of imbalance or compromised tube integrity.

Aliquot Serum

22m

- 4 Carefully remove SSTs from the centrifuge and inspect for separation of red blood vs serum layers.

2m

Transfer SSTs to a sanitized laminar flow hood.

- 5 Using a P1000, carefully aliquot serum into 2D barcoded tubes at **500µL each**.

20m

Return 2D tubes with serum to the rack and place in the -80C freezer.

- 6 Recap and freeze the SST tubes to preserve DNA in the red blood cell pellet