

Jan 14, 2026

DNA extraction with Modified GenoLyse kit

DOI

<https://dx.doi.org/10.17504/protocols.io.n92ld1myol5b/v1>

Hubert Senanu¹, Micheal Frimpong¹

¹Kumasi Centre for Collaborative Research, Kwame Nkrumah University of Science and Technology



Hubert Senanu

Create & collaborate more with a free account

Edit and publish protocols, collaborate in communities, share insights through comments, and track progress with run records.

Create free account

OPEN  ACCESS



DOI: <https://dx.doi.org/10.17504/protocols.io.n92ld1myol5b/v1>

Protocol Citation: Hubert Senanu, Micheal Frimpong 2026. DNA extraction with Modified GenoLyse kit. **protocols.io**
<https://dx.doi.org/10.17504/protocols.io.n92ld1myol5b/v1>

License: This is an open access protocol distributed under the terms of the **[Creative Commons Attribution License](#)**, which permits unrestricted use, distribution, and reproduction in any medium, provided the original author and source are credited

Protocol status: Working

We use this protocol and it's working

Created: January 13, 2026



Last Modified: January 14, 2026

Protocol Integer ID: 238535

Keywords: dna extraction with modified genolyse kit, bacterial dna from patient specimen, dna extraction, efficient dna extraction, bacterial dna, modified genolyse kit, genolyse kit, dna, test procedure, extraction, whole test procedure, isolate, specimen, simple procedure, procedure, patient specimen, standard operating procedure, culture sample

Abstract

This document describes the standard operating procedure for a fast and easy way to isolate bacterial DNA from patient specimens and culture samples using the GenoLyse kit. The simple procedure allows for a fast and efficient DNA extraction in less than 20 minutes. Since the DNA extraction only takes a short amount of time, the whole test procedure is reduced. The procedure requires minimal instrumentation.

Materials

- Hain Genolyze kit (Hain, Germany)
- Phosphate buffer solution
- 1.5 ml/ 2 ml DNase/RNase free centrifuge tubes (Eppendorf, Germany)
- 100 µl, 1000 µl pipette filter tips, DNase/Rnase free (Biozym, Germany).
- Mini-Centrifuge
- Dry block heater
- Vortexer
- Tube racks



General considerations

- 1 This document describes the standard operating procedure for a fast and easy way to isolate bacterial DNA from patient specimens and culture samples using the GenoLyse kit. The simple procedure allows for a fast and efficient DNA extraction in less than 20 minutes. Since the DNA extraction only takes a short amount of time, the whole test procedure is reduced. The procedure requires minimal instrumentation.

Preparations

- 2 GenoLyse kits should be stored at 2-8 °C. All other solutions could be stirred at room temperature.

Procedure

- 3 Elute samples into GenoLyse buffer (250 µl for swabs or 100 µl for FNA).
Vortex vigorously for 20 sec.
Incubate samples for 10 min at 95 °C (Remove swabs before incubation).
Add equal volume of neutralization buffer.
Centrifuge at high speed for 5 mins.
The extracted DNA (supernatant) may be directly used for amplification or stored at -20 °C.

Protocol references

Frimpong M, Ahor HS, Sakyi SA, Agbavor B, Akowuah E, Phillips RO. Rapid Extraction Method of Mycobacterium ulcerans DNA from Clinical Samples of Suspected Buruli Ulcer Patients. *Diagnostics (Basel)*. 2019;9(4). Epub 20191126. doi: 10.3390/diagnostics9040204. PubMed PMID: 31779247; PubMed Central PMCID: PMCPMC6963521.