

May 10, 2019

U Mass - Cholesterol (HDL)

DOI

dx.doi.org/10.17504/protocols.io.xt6fnre



Jason Kim¹

¹University of Massachusetts

Mouse Metabolic Phenotyping Centers
Tech. support email: info@mmpc.org



Lili Liang

Create & collaborate more with a free account

Edit and publish protocols, collaborate in communities, share insights through comments, and track progress with run records.

Create free account

OPEN  ACCESS



DOI: dx.doi.org/10.17504/protocols.io.xt6fnre

External link: <https://mmpc.org/shared/document.aspx?id=170&docType=Protocol>

Protocol Citation: Jason Kim 2019. U Mass - Cholesterol (HDL). protocols.io
<https://dx.doi.org/10.17504/protocols.io.xt6fnre>

License: This is an open access protocol distributed under the terms of the [Creative Commons Attribution License](#), which permits unrestricted use, distribution, and reproduction in any medium, provided the original author and source are credited

Protocol status: Working

We use this protocol and it's working

Created: February 06, 2019

Last Modified: May 10, 2019

Protocol Integer ID: 20062

Keywords: HDL-cholesterol, cardiovascular disease, serum levels of hdl, cholesterol, cholesterol metabolism, hdl, using roche cobas clinical chemistry analyzer, roche cobas clinical chemistry analyzer, cardiovascular disease, spectrophotometric measurement, serum level

Abstract

Summary:

This experiment involves a spectrophotometric measurement using Roche Cobas Clinical Chemistry Analyzer. Serum levels of HDL-cholesterol reflect cholesterol metabolism and are associated with cardiovascular disease.

Materials

MATERIALS

⊗ HDL Cholesterol Plus 3rd gen Roche Catalog # 05401488 190

⊗ Calibrator f.a.s. Lipids Roche Catalog #12172623 160

⊗ Precinorm L Roche Catalog #10781827 122

⊗ Precipath HDL/LDL-C Roche Catalog #11778552 122

⊗ NaCl Diluent 9% Roche Catalog #04774230 190

⊗ Cleaner Roche Catalog #04774248 190

⊗ Micro Sample cups Roche Catalog #11406680 001

⊗ NERL High Quality Water Fisher Scientific Catalog #9805

Note:

Roche, [RRID:SCR_001326](#)

Fisher Scientific, [RRID:SCR_008452](#)



Before start

Notes:

- ✓ Try to use freshly prepared serum and plasma samples for this assay.
- ✓ No dilution or treatment of the sample is required, but plasma samples should be centrifuged to remove any fibrin/fibrinogen clumps.
- ✓ Samples should be stored at 2-8°C for 24 hours prior to analysis. For longer periods, store samples at -70°C, and avoid repeated freeze/thaw cycles.
- ✓ A 50 µl dead volume is required in addition to sample volume for multi-protein analysis (typically 1-5 µl).



- 1 Perform daily quality control assessment of instrumentation before analysis.
- 2 Load each sample into a specialized micro-sample cup for the clinical chemistry analyzer.
- 3 Select Cholesterol (HDL) test on display and run the analysis.
- 4 Collect and analyze the data.